

Link Your Scanner with Sun Dental Labs UK

Free digital Model, Free Shipping, Free Implant Scan Bodies

3Shape

- Go to <https://us.3shapecommunicate.com>
- Select "CONNECTIONS"
- Request a connection with Sun Dental Labs UK– using email address eucad@sundentallabs.com
- After your connection is approved, select Sun Dental Labs when sending files

iTero

- Please contact your iTero representative and request a link to Sun Dental Labs UK– you can use email address info@sundentallabs.co.uk

Dentsply Sirona / DS Core

- Log in using the 'Connect Case Centre' software
- Choose 'My Favourite Recipients' from the left-hand menu
- Search for Sun Dental Labs UK and click add – you can use email address eucad@sundentallabs.com

Carestream / Dexis IS connect

- Visit "Carestream " or 'Dexis IS connect' on your scanner
- Search for Sun Dental Labs UK or you can use email address to find us - cadcam@sundentallabs.co.uk
- Add Sun Dental Labs UK
- Select Sun Dental Labs UK when submitting scans

Shinning 3D

- Dentist can add us via Shining 3D online portal within the "Connection" tab – add Sun Dental Labs UK– or find us using email address info@sundentallabs.co.uk

Medit

- Log into www.meditlink.com with the administrator account
- Go to 'Partners on the left panel'
- Search for Sun Dental Labs UK or find us using email address info@sundentallabs.co.uk and click the 'Request Partnership' button
- You'll see that the Sun Dental Labs UK has been added to the "My Partner List"

For other scanners please send files to info@sundentallabs.co.uk via

<https://wetransfer.com>

Do you have a question about our products or services? Need help on submitting a case or creating a MySun Account? Our team is happy to help! Simply give us a call to talk to a friendly Sun Dental Customer Service Representative on 01226 786163 or visit our website <http://www.sundentallabs.co.uk/>

